

Instructions for use



Sanquin Reagents B.V.
Plesmanlaan 125
1066 CX Amsterdam
The Netherlands

Phone: +31 20 5123599
Fax: +31 20 5123570
Reagents@sanquin.nl
www.sanquin.org/reagents

Cellbind incubator

REF K7314

IVD CE

090_v01 09/2019 (en)

For professional use only

Quick start reference manual for Cellbind incubator

The instrument should only be used as intended, in perfect technical condition, by qualified persons and under strict observation of current safety and accident prevention standards. For detailed instructions please refer to the Allsheng K37-24 User Manual and IFU of products mentioned below.

General information

Cellbind incubator is used to incubate Cellbind micro column cards. The use of PeliControl (REF K1379) is required at least once every day when using the system. In the event of failure to pass the quality control tests, the reason for this must be identified. The laboratory tests should not be considered reliable until the problem has been resolved.

Precautions

See IFU of products mentioned above.



Improper use and handling can result in danger of life and limb of the user or third parties and impairment to the machine or to other material assets. The machine should only be used for the specified applications and only in an impeccable technical safety condition.

Reagents

The following micro column cards can be used in combination with Cellbind incubator:

Cellbind Screen	REF K7000	CE
Cellbind Direct	REF K7011	CE
Cellbind Direct Type	REF K7012	CE

Operation guide

- Power**
Power on the main switch. The incubator is ready to operate when the display becomes visible.
- Control elements**

- Setting the timing time**
Press "▲" or "▼" to set the timing time from 1 sec to 99 min 59 sec.
When pressing "▲" or "▼", the unit digit will flicker, and the time can be modified. After 6 seconds, the flicker will disappear, exiting the setting interface.
- Setting the temperature**
The default setting is 37.0°C.

- Start and Stop**
The instrument will start to heat automatically according to the set temperature. The left "START/STOP" key controls the left block and the right "START/STOP" key controls the right block.
- When the temperature of either block reaches 37.0°C, put the Cellbind card(s) into the block, then close the lid and press the "START/STOP" key to start the timing of the block.
- When the timer reaches 0, the alarm will sound and the display shows "OVER". Press any key to return, or press the "START/STOP" key to restart the timer.

Operation guide

8 Press "START/STOP" key for 1 second to stop the timer.

Interpretation

See IFU of products mentioned above.

Limitations

See IFU of products mentioned above.

Sanquin products are guaranteed to perform as described in the original manufacturer's instructions for use. Strict adherence to the procedures, test layouts and recommended reagents and equipment is essential. Sanquin declines all responsibility arising from any deviation thereof.